



Frequently Asked Questions

Where does the water come from?

Shower Bay's unique quick-connect system easily snaps onto most home faucets. The water travels from the faucet through an in-hose to the Shower Bay unit.

Where does the water go?

The built-in pump located at the base of Shower Bay removes the water through the attached out-hose to a nearby sink or tub drain.

Is Shower Bay supposed to fit in my bathroom?

Shower Bay may be used in a bathroom, but it was also designed to work in any room of the home. We understand that most bathrooms are not large enough for adequate wheelchair maneuverability. In most cases Shower Bay is set up in the corner of a bedroom or spare area of the home, such as a den, laundry room or garage.

Do I need a contractor, plumber or permits?

Because there is no installation, a contractor, plumber or building permit is not required.

Does Shower Bay fill up like a bathtub?

Shower Bay is a shower and does not fill up like a bathtub.

View more FAQ's online at www.ShowerBay.com



Contact Information

Australian Distributer:



16-20 Ragless Street, St Marys SA 5042

T: 08 8374 1099

office@hiline.net.au

www.hiline.net.au

www.ShowerBay.com



Made in U.S.A.



Shower Safely Without Remodeling

www.ShowerBay.com



What is Shower Bay?

SHOWER BAY is a safe and portable shower designed for wheelchair users. Unlike other solutions on the market, Shower Bay offers a true shower experience without requiring dangerous wet-environment transfers or expensive home renovations. The unique design allows for quick assembly in any room of the home.

Safety First

At Forward Day we believe that a shower should be enjoyed, not dreaded. Your daily shower activity should never be missed due to anxiety over a dangerous transfer or possible accident.

Family members and caregivers will also gain peace of mind with Shower Bay. Assisting to bathe someone with mobility limitations can be extremely stressful and unsafe. It is not uncommon for acting caregivers to strain their own body or even slip while helping the user in or across a bathtub or shower. Shower Bay eliminates all wet-environment transfers and allows for transfers to be conducted where both the user and caregiver feel most comfortable and confident—at bedside or chair-to-chair.

Safe

The absolute safest way to shower is rolling completely into the shower without any wet-environment transfers.

Clean

Makes showering every day achievable, rejuvenating the body and mind.

Easy to Use

Quickly assembles without any tools or need of a contractor and plumber.

Shower Bay is a Great Option for:

Home Owners

Those wanting to avoid a costly and time-consuming bathroom remodel

Renters

Those who generally cannot modify or remodel their apartment or home

Aging in Place

Families seeking to avoid the high costs of assisted living facilities and keep their loved one at home

First Floor Living Restrictions

Homes with only a half-bath available on the first floor

Temporary Living Situations

Families that have a loved one with mobility limitations temporarily stay at their home

Injury Rehab Patients

Individuals with the need for a temporary bathing solution during the rehabilitation process

Residential Moves

Users planning to change residences will benefit from the one-time investment in a portable shower

Equipment Rentals

Those looking to rent the shower for time or cost reasons

Showering made Simple!